# CONCARE® HY-LUB

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For dry, tired or strained eyes

### Description

CONCARE® HY-LUB Moisturizing Drops is a viscoelastic, hypotonic and sterile solution. It contains a hyaluronic acid biomolecule acting as a moisturizing and lubricant agent. The product is preservative-free and compatible with contact lenses.

### Indications

Thanks to the properties of hyaluronic acid, CONCARE® HY-LUB Moisturizing Drops protect and lubricate the surface of the eye, providing long-lasting relief from the symptoms of dry eye caused by:

- keratoconjunctivitis sicca (Sjogren's syndrome)
- wearing contact lenses
- exposure to air conditioning, wind, cold and dry environments, dust, smoke or pollution
- using computer screens or travelling by car or plane for long periods of time
- swimming in pools treated with chlorine

### Dosage

Use the product as often as advised by your eye care specialist. Depending on symptom severity, you may need to apply the product 4 to 8 times per day.

### Method of use

- Remove the safety seal on the cap before using the product for the first time.
- Remove the cap.
- Squeeze both sides of the bottle with your fingers until a drop forms. Instill the drop directly into the eye. For contact lens users, apply the product before putting in contact lenses to make them easier to insert and more comfortable to wear.
- After use, place the cap back on the bottle. Discard the product within three months after opening the bottle for the first time.

### Caution

- Store the product at a temperature of 2 to 25 °C. Avoid extended exposure to light.
- Do not use CONCARE HY-LUB Moisturizing Drops if the seal on the cap or the bottle are damaged.
- When using the product, take care not to let the bottle dropper touch your eye.
- Do not touch the bottle dropper with your fingers or any other surfaces to avoid contaminating the product.
- If you are taking medication, check with your doctor before using CONCARE® HY-LUB Moisturizing Drops for precautionary reasons.
- In case of persistent irritation, refrain from using the product and talk to your eye care specialist. You may be allergic to one or more of its components.
- After use, your visual acuity may decrease momentarily. If this happens, do not drive or handle heavy machinery until your normal eyesight is restored.
- Write down the date when the bottle is first opened.
- Discard the product within three months after opening the bottle.
- Keep out of the reach of children.

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